


## Dräger DrugCheck® 3000 On-the-spot Drug Screening Device

Use the Dräger DrugCheck® 3000 to find out within minutes if a person has recently consumed certain drugs. This compact oral fluid-based drug screener yields reliable results hygienically, quickly and easily. The device does not require electricity and can be used practically anywhere.

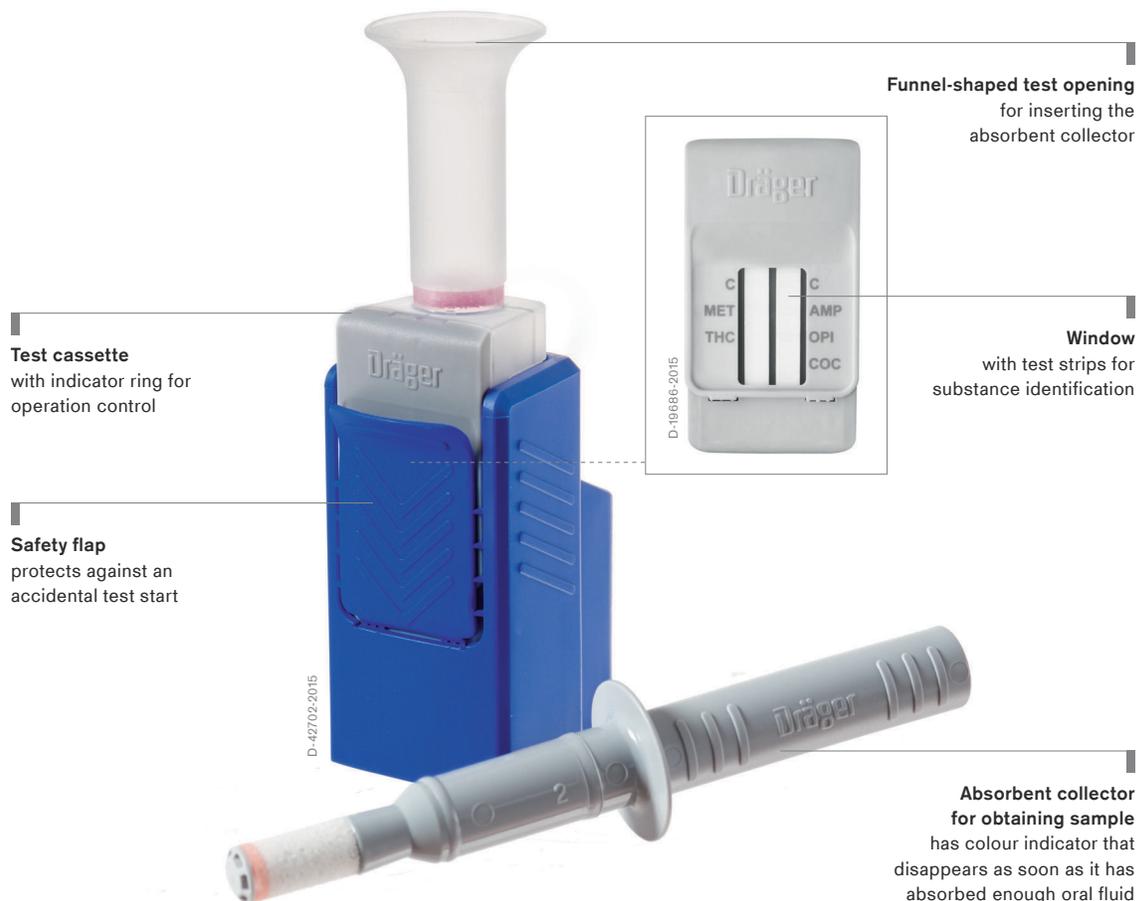

## Benefits

---

### Obtain samples easily and safely

The Dräger DrugCheck® 3000 consists of two components: an absorbent collector (swab) for obtaining an oral fluid sample and a test cassette for the analysis. The test cassette contains buffer liquid and a window with two test strips that display the control and test lines. The test can be performed in a few easy steps: 1) take a saliva sample using the swab, 2) insert the swab into the funnel opening of the test cassette, 3) shake the test kit, 4) begin the test analysis, 5) read the results shown in the display window. A colour indicator on the swab disappears as soon as it has absorbed enough oral fluid for a test.

As soon as the control lines appear in the window, you can read the results. If the test result is negative, a line will appear alongside the respective substance class (drug). This means that none of the target substance was detected in the sample. If a line fails to appear next to one of the substance classes, then the result for this substance is positive.

---

### Fast and sensitive analysis

You can simultaneously check an individual for up to five substance classes with the Dräger DrugCheck 3000: cocaine, opiates, amphetamines, methamphetamines/designer drugs (e.g. Ecstasy, MDMA), and cannabis (THC). Of all these substance classes, cannabis is the drug consumed most frequently - and also the most difficult of all compounds to identify (THC = delta-9- tetrahydrocannabinol). This is why Dräger optimised the DrugCheck 3000 to detect THC - and now offers two measurement options. The fast option (25ng/ml THC threshold) takes less than 3 minutes and the sensitive option (15ng/ml THC threshold) takes less than 5 minutes to show a positive or negative result on the test window.

---

### Convenient, hygienic on-the-spot drug detection

The pocket-sized design makes the DrugCheck 3000 easy to transport. It can be made ready for use quickly and easily, and is suitable for use in safety sensitive industries. The DrugCheck® 3000 test kit is tamper-proof and hygienic to use.

---

### Maintenance Free

Because the DrugCheck 3000 has no electrical parts, it's safe to use in hazardous areas. The portable test kit can be easily disposed of after use and there is no calibration or servicing required - which makes using the device a hassle-free experience.

---

### Backed by decades of experience

Dräger has decades of experience in alcohol measuring equipment and drug detection methods. For the DrugCheck® 3000, Dräger employed the testing principle of the proven Dräger DrugTest® 5000 system, which reliably detects even minute traces of THC.

## Infographic

**Dräger DrugCheck® 3000**  
2 Measurement Options for Detecting THC

<b>FAST</b>	<b>SENSITIVE</b>
	
25ng/ml	15ng/ml
Results in < 3 minutes	Results in < 5 minutes

When using the test kit, please reference the Quick IFU document for the correct procedure of detecting the desired THC threshold.

## Related Products



D-54720-2012

### **Dräger DrugTest® 5000**

No pipetting, no drips, no timing: With the Dräger DrugTest® 5000, drug test is carried out simply and quickly. The collected saliva sample can be analyzed immediately—for accurate results on the spot.

## Related Products



---

### Dräger SSK 5000

Ideal for surface drug testing applications, the Dräger SSK 5000 sampling system is quick and easy to use. When used with the Dräger DrugTest 5000, the sample can be transferred to a test cassette and screened for possible drug contamination. The sample can also be stored and transported for further testing.



---

### Dräger Alcotest® 3820

The Dräger Alcotest® 3820 eliminates guesswork by providing a reliable breath alcohol concentration reading every time. The device ensures precise results by using the same measurement technology used by police in over 30 million breath alcohol tests a year.



---

### Dräger Alcotest® 5000

The Dräger Alcotest® 5000 is a professional breath-alcohol tester which detects the presence of alcohol. The high-speed breathalyser lets you perform numerous tests in minimal time. Its special funnel reduces the back-flow of expired air to a minimum, preventing the risk of infection to subsequent test subjects.

## Technical Data

### Test for detecting drugs in oral fluid

Dimensions (W x H x D)	32 x 111 x 57 mm 1.26 x 4.37 x 2.24 inches
Weight	<30 g (0.07 lb)
Operating range	Operation: 5 °C to 40 °C (41 °F to 104 °F) at 5% to 95% relative humidity
Storage/Transportation	4 °C to 25 °C (39 °F to 77 °F)
Time of measurement	Fast measurement: analysis <3 minutes Sensitive measurement: analysis <5 minutes
Selecting the measurement mode	Depending on the selected preincubation time and the desired THC cutoff, fast or more sensitive reading.
Licenses/Standards	Outside of the European Union for non-medical use (Non-IVD) or for forensic use (only USA).

## Ordering Information

Dräger DrugCheck® 3000 Test for detecting drugs in oral fluid	Description	Ordering no.
Dräger DrugCheck® 3000 STK 5 Non-IVD	Packaging unit 20 kits	83 25 580
Dräger DrugCheck® 3000 detects the following substances:		
<ul style="list-style-type: none"> <li>- Cocaine (COC)</li> <li>- Opiates (OPI)</li> <li>- Cannabis (THC)</li> <li>- Amphetamines (AMP)</li> <li>- Methamphetamines (MET)</li> </ul>		

## Notes

Not all products, features, or services are for sale in all countries.  
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to [www.draeger.com/trademarks](http://www.draeger.com/trademarks) to find the current status.

### **CORPORATE HEADQUARTERS**

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Allemagne  
[www.draeger.com](http://www.draeger.com)

### **CANADA**

Draeger Safety Canada Ltd.  
2425 Skymark Avenue, Unit 1  
Mississauga, Ontario, L4W 4Y6  
Tél +1 905 212 6600  
Toll-free +1 877 Drager 1  
(+1 877 372 4371)  
Fax +1 877 651 0902  
Toll-free Fax +1 800 329 8823

Locate your Regional  
Sales Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

